

## FREQUENTLY ASKED QUESTIONS

Q. Is Acryline 2 similar to Denturite Sea-Bond?

A. Yes it is. It also a liquid and powder that you mix together and apply to the denture. Also:

- 1) It has two applications contained within the kit that can be used for the upper AND/OR the lower dentures.
- 2) It is a temporary cushiony action that can last for up to 2 years.
- 3) Acryline 2 reliner's firm custom-fit lets you eat anything right away. It keeps food from getting under plates.
- 4) Acryline 2 is for temporary refitting of dentures between visits to the dentist.

Q. What is Acryline 2 made from?

A. The active ingredients in Acryline 2 soft denture relining kit are: liquid-Ethyl Alcohol and 2-Ethylhexyl Diphenyl Phosphate Powder – Benzoyl Peroxide, Cadmium Pigments, Ethyl Methacrylate, Titanium Dioxide.

The technical bulletin listing all the ingredients as well as the application techniques can be found at [www.acryline2tb.com](http://www.acryline2tb.com)

Q. Why should I use Acryline 2 soft denture reliner kit?

A. Many consider a soft denture reliner the best type of reliner available. It can last up to 2 years. Doing a soft reliner at home with a soft denture reliner kit is by far the least costly route to relief from loose and irritating dentures.

Q. Who is the manufacturer of Acryline 2 denture reliner kit? Is it made in the USA?

A. Acryline 2 soft denture reliner kit is manufactured by Lee Pharmaceuticals in South El Monte, California, 15 miles east of Los Angeles.

Lee Pharmaceuticals is a FDA registered manufacturer. They began in the 1960s as a dental manufacturer by first obtaining government research contracts. They then received a contract with Johnson & Johnson to develop a composite filling material to fill your cavities at the dentist's office. This product had similar strength as the old silver (amalgam) fillings but was a translucent (clear) material which appeared white. This was due to the adjacent colors being "absorbed or picked up" by the clear filling material. The company had developed its expertise with the help of the founder Dr. Henry L. Lee Jr., who co-authored 2 books that were considered the definitive books on epoxy resins. (The

textbook of Epoxy Resins and The Handbook of Epoxy Resins – both published by McGraw Hill). These new generation of dental Polymers were based on epoxy resins which had been developed in the 1930s by a Swiss firm Ciba-Geigy. Lee Pharmaceuticals also manufactured nail extension products like Lee Nails and Lee Sculptured Nails and Lee Press On Nails that were based on their knowledge of adhesives technology. In 2002 Lee Pharmaceuticals acquired the product Acryline 2 from Numark Laboratories. Acryline 2, first introduced in 1956, is one of the leading denture reliner products available on the market today.

Q. Is Acryline 2 denture relining kit come with a guarantee?

A. Yes, Acryline 2 is covered by a 30 day money back guarantee. Just send the product back to Lee Pharmaceuticals for a full refund.

P.O. Box 3836  
Dept. R  
El Monte, CA 91733

Q. Does Acryline 2 contain zinc?

A. No, Acryline 2 denture reliner kit is ZINC FREE!

Q. Will Acryline 2 denture relining kit work on an upper denture?

A. Yes, it will work on an upper denture as well as lower dentures. It can also be used on partial dentures. Acryline 2 will adhere to acrylic, plastic, and porcelain partials only. It should not be used on metal.

Q. How do I clean my dentures with a reliner from Acryline?

A. You can still use effervescent tablets like efferdent or Polident. We recommend you utilize a shallow dish to soak just the teeth side of the denture in the solution. Please try to avoid getting the actual liner portion wet from the solutions. You can still soak the denture in water overnight. Then use toothpaste or denture paste to clean the liner along with a soft toothbrush.