

Acryline[®] 2

INGREDIENTS:

Liquid – Ethyl Alcohol, 2-Ethylhexyl Diphenyl Phosphate
Powder – Benzoyl Peroxide, Cadmium Pigments, Ethyl Methacrylate, Titanium Dioxide.

False Teeth Break?

Good news! Repair them at home with
PLATE-WELD



For Emergencies,
PLATE-WELD
temporarily repairs breaks,
cracks and replaces teeth
at home. Use as directed.
Get **PLATE-WELD** now
at your drug store counter.

Distributed by
LEE PHARMACEUTICALS
1434 Santa Anita Ave.
South El Monte, CA 91733
(800) 950-5337
Made in U.S.A.

#179011C
Rev. 5/10

Acryline[®] 2

TEMPORARY DENTURE RELINER INSTRUCTION SHEET

For Plastic
and Rubber Dentures

CONGRATULATIONS!

You have just purchased a remarkable product, ACRYLINE 2, which has given thousands of denture wearers temporary relief from loose, troublesome dentures.

Why is there so little liquid and powder?

Each portion is sufficient material to do a complete denture reliner. A thin film applied is all that is required for suction and fit.

Will Acryline 2 work on rubber or metal plates?

Acryline 2 works on rubber plates. Acryline 2 will not work on metal plates. Because everybody's taste buds vary, some users may notice a slight bitter taste after applying Acryline 2. This is only temporary and occurs only during setting time. If this is noticed, simply soak denture in a solution of baking soda and water for approximately fifteen minutes, then rinse thoroughly.

HOW TO REMOVE ACRYLINE 2

To remove Acryline 2, soak for 10 minutes in hot water, peel or scrape out the liner with a sharp knife blade. Heating the blade aids removal!

HOW TO SLOW DOWN OR SPEED UP THE SETTING TIME OF ACRYLINE 2

- (A) To slow the setting of ACRYLINE 2, cool the liquid in the refrigerator before mixing. (The cooler the liquid the slower the setting time.)
- (B) To speed up the setting of ACRYLINE 2, warm the liquid vial by immersing in warm water before mixing. (The warmer the liquid the faster the setting time.) Be careful not to get it too warm as it might set too fast to work with.

Each vial of powder and each vial of liquid together contain enough for 1 reliner. Reliner only 1 plate at a time. Do not reliner both at once.



1 Clean and dry plate thoroughly

Thoroughly clean denture with a brush and any good cleaner. Rinse and dry. Make sure denture is completely dry before next step.

HOW TO MAKE IT EASIER TO REMOVE EXCESS ACRYLINE 2 FROM YOUR DENTURE LATER.



4 Fit denture in mouth

Place the denture in your mouth. Make sure it is seated properly. Bite down slowly, firmly, gently on opposing teeth making sure your plate meets in correct position. Hold in this position for 3-5 minutes.

meets in correct position. Hold in this position for 3-5 minutes.

After Step #1 of instructions:

- (A) Apply a light coat of Petroleum Jelly to the outside of your dentures. Coating the outside helps remove any excess ACRYLINE 2 that might run on outside.
- (B) With a clean cloth wipe off all excess Petroleum Jelly.

Proceed to Step #2 of instructions.



2 Pour and Stir Acryline 2

Pour one vial of liquid and one vial of powder into mixing cup.

IMPORTANT: Stir immediately and quickly only until smooth (about 8 or 9 seconds). Follow next step immediately.



5 Remove and trim excess

Remove denture from mouth and rinse with cold water. Trim away any excess material

which may have squeezed over the edges, using a sharp knife.



Acryline 2

(RELINER ONLY
ONE DENTURE AT A TIME)

- (A) LOWER DENTURE: Immediately pour and spread the mixed ACRYLINE 2 into the ridge hollow as shown in diagram.
- (B) UPPER DENTURE: Immediately pour and spread the mixed ACRYLINE 2 around the ridge hollow area and along the rear edge of the plate as shown in the diagram.

IMPORTANT: Use of these preparations or devices may temporarily decrease the discomfort; however, their use will not make the denture fit properly. Special training and tools are need to repair a denture to fit properly. Dentures that do not fit properly cause irritation and injury to the gums and faster bone loss, which is permanent and may require a completely new denture. Changes in the gums caused by dentures that do not fit properly may require surgery for correction. Continuing irritation and injury may lead to cancer in the mouth. You must see your dentist as soon as possible.

CAUTION: Liquid, Combustible. Keep away from heat, sparks and open flame.

WARNING: FOR TEMPORARY USE ONLY. Check your relined plate with your dentist promptly. Ask him to make sure it fits properly. He may recommend a permanent relining that only his training and equipment can provide. Long-term use of an improperly fitting denture may lead to faster bone loss, continuing irritation, sores and tumors.
FOR USE ONLY UNTIL A DENTIST CAN BE SEEN.